


Material File Update

3Shape Dental Manager

Import Material File (1/5)

Attention: If you are using individual settings, please do always perform a backup before the update. You can do this for example with a copy of your 3Shape configuration folder on the hard disk C:/.

1. Open the **Control Panel (1)**.

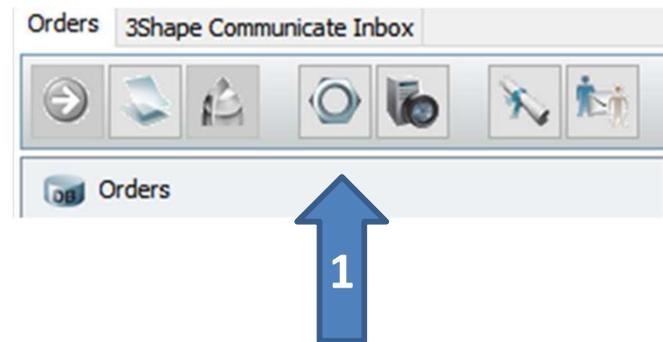


2. Select **Manufacturers (2)**.



Import Material File (2/5)


3. Choose **Kulzer Milling Center (1)** and click on **Download materials (2)**.

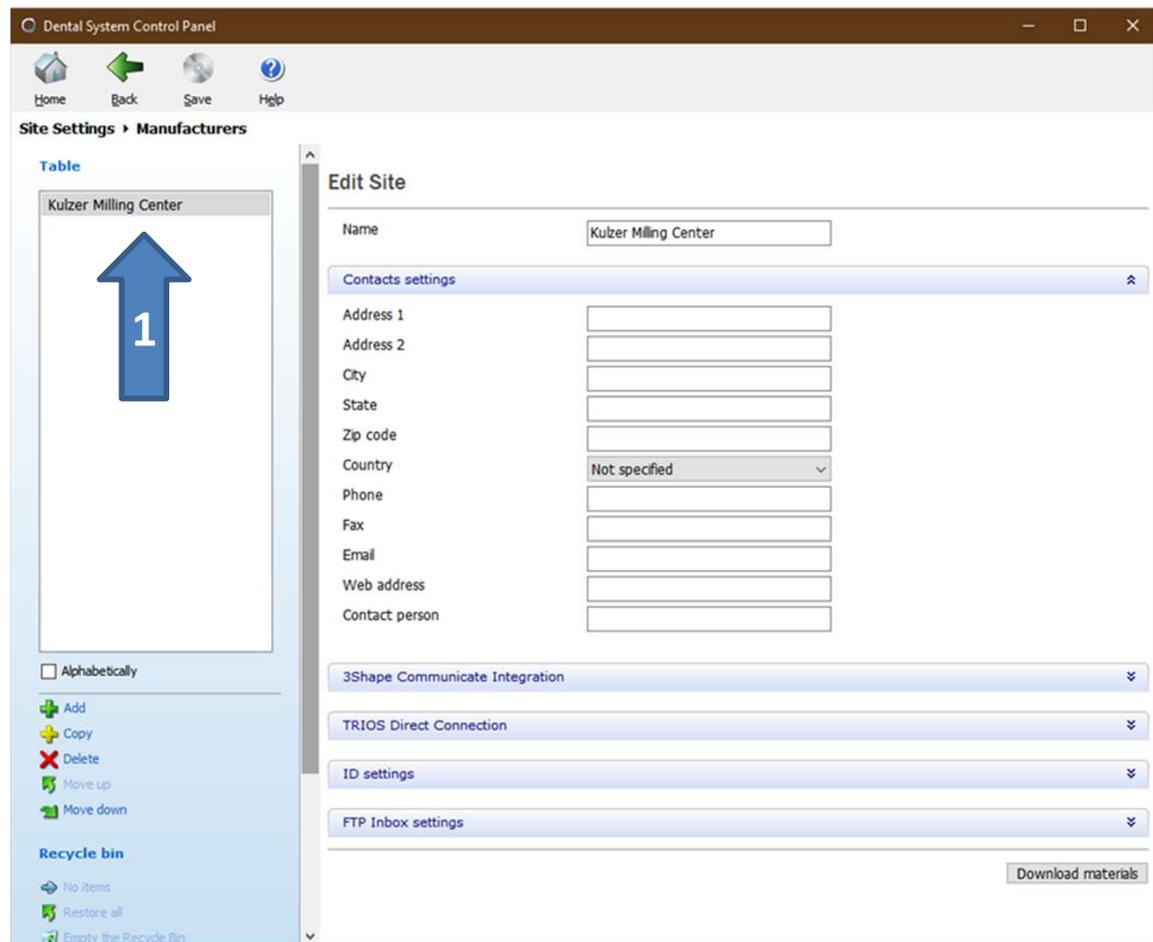


The screenshot shows the 'Dental System Control Panel' window. The main content area is titled 'Site Settings > Manufacturers'. On the left, there is a table with one entry: 'Kulzer Milling Center'. A blue arrow with the number '1' points to this entry. Below the table are several action buttons: 'Alphabetically', 'Add', 'Copy', 'Delete', 'Move up', and 'Move down'. On the right, the 'Edit Site' form is visible, containing various input fields for 'Name', 'Address 1', 'Address 2', 'City', 'State', 'Zip code', 'Country', 'Phone', 'Fax', 'Email', 'Web address', and 'Contact person'. The 'Name' field is filled with 'Kulzer Milling Center'. Below the form are several expandable sections: '3Shape Communicate Integration', 'TRIOS Direct Connection', 'ID settings', and 'FTP Inbox settings'. At the bottom right of the form, there is a 'Download materials' button, which is pointed to by a blue arrow with the number '2'.

Import Material File (3/5)

4. Click on **Import**.

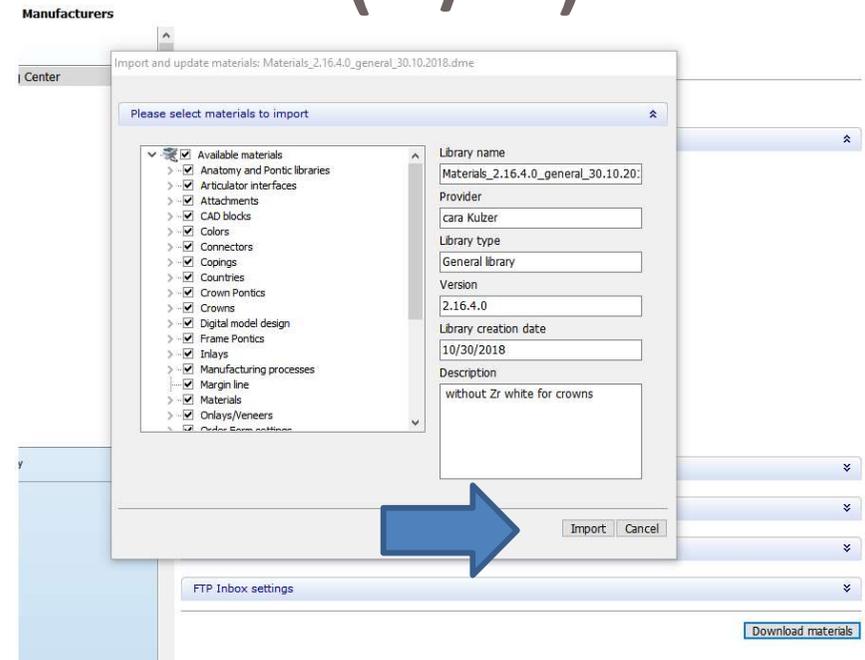


5. Check the box **Do the same for all similar cases (1)**.
Then click **Update (2)**.
The system is working for a few minutes now.

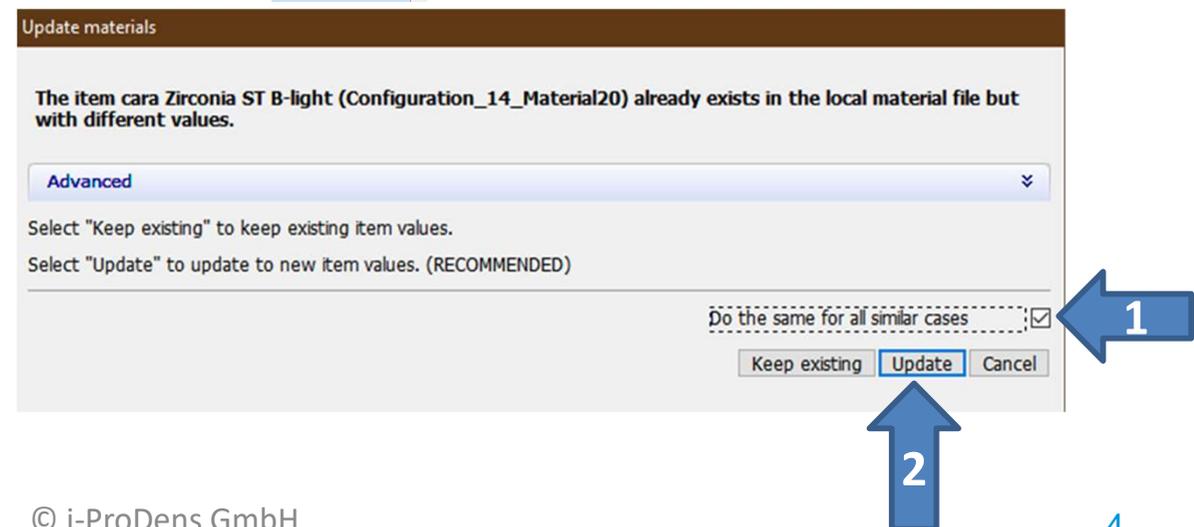

Import Material File (4/5)

6. The request to delete the no longer available material i-PD PMMA medium follows. Click “Yes, all”.
7. If the import process was successful please click **OK**.
Close the Control Panel and click „Yes“ to save the data.

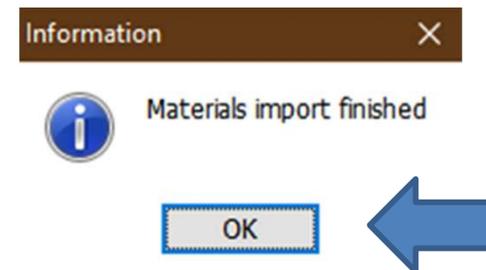

Import Material File (5/5)

8. The manufacturer **Kulzer Milling Center** has been changed to **i-ProDens Laser/ Milling Center**.

